

**Optional carry strap and EZ Mounting Plate #1604
sold separately**

7 Level Communication Builder #7075

USER'S GUIDE



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-832-8697

customer_support@enablingdevices.com



50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

It grows with you!

The 7-Level Communication Builder is a self-contained communication device. It allows the user to record and play back 1, 2, 4, 8, or 16 different messages per level. There are seven levels for recordings, giving you a total of up to 112 messages (in the 16 window setting). Total recording time is 300 seconds. The unit comes equipped with 5 sturdy frame/overlays in 1, 2, 4, 8 and 16 windows. Re-recording is required when you change window frames. Frame size is 8½" x 7". Time per level by window setting, using all seven levels:

1 Window setting = 42 seconds per segment per level

2 Window setting = 21 seconds per segment per level

4 Window setting = 10½ seconds per segment per level

8 Window setting = 5¼ seconds per segment per level

16 Window setting = 2½ seconds per segment per level

Size: 8 ¼"L x 10 ¾"W x 3 ¼"H. Requires 4 AA Batteries. Weight: 2¼ lbs.

OPERATION:

1. Carefully turn the device over to reveal the battery compartment. A small #1 Phillips head screw must be removed from the battery compartment cover in order to remove it. Observing proper polarity, install four AA alkaline batteries Use alkaline batteries only (e.g. Duracell or Energizer brand). Do not use heavy duty, rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. Never mix old and new batteries together or different brands or types together. Replace battery compartment cover and screw.
2. Place your icon grid over the gray message pad area then install the appropriate window frame. Next turn your window frame selector knob to the appropriate setting. Please note pre-made grid layouts can be made with our Easy To Use Print it Icon Maker item No.4069X or Tobii-Dynavox's Board Maker program.
3. Turn the unit on by turning the "ON/OFF/VOLUME" knob into the "ON" position. You will hear a slight audible click as you turn the knob.
4. The microphone is located on the back of the unit next to the red "RECORD" button. To record, depress and hold the "RECORD" button first, then depress one of the message pads simultaneously and speak into the microphone (your mouth should be approximately 4-6" from the microphone). Release both buttons once recording is over. Playback your message by pressing and releasing the same pad used to record. Each of the remaining message pads can be "programmed" in the same manner. (The unit will retain recorded information indefinitely, even if the batteries are disconnected. Previously stored information will be erased only when a new recording is performed on the pad.)

5. Adjust playback volume by rotating the on/off "VOLUME" control knob to desired level. Turn the unit off by rotating the "VOLUME" control knob counterclockwise until you hear an audible "click". Using the "LEVEL" knob, select level 2. Repeat steps 2 and 3. Repeat procedures for remaining levels.

IMPORTANT NOTES:

This unit has a total record time of 300 seconds. The breakdown is as follows. Time per level by window setting.

- 1 Window setting = 42 seconds per segment per level
- 2 Window setting = 21 seconds per segment per level
- 4 Window setting = 10.5 seconds per segment per level
- 8 Window setting = 5.25 seconds per segment per level
- 16 Window setting = 2.5 seconds per segment per level

The 7 Level Communication Builder

- will shut off automatically after playback to preserve battery life. However, this device still drains a very small current in this "sleep" mode. Turn the power off when not in use. Doing this can extend overall battery life. Batteries should be removed during long time storage as they may leak and damage the unit.
- The unit may be wiped clean using mild detergent and water on a damp cloth. **Do not submerge the unit in water.**

TROUBLESHOOTING:

If the unit fails to operate, or operates incorrectly, try the following suggestions:

- ◆ Disconnect the power by removing all of the batteries for 20 seconds to reset the unit. If the unit still does not work correctly after batteries are reinstalled replace with new alkaline batteries. **Make sure proper polarity is observed or damage to unit may occur.**
- ◆ Make sure unit is on and volume is set at an audible level.
- ◆ Try to record again to see if the unit will "accept" the new recordings.
- ◆ **Master Reset (Erase All Messages):**

To clear all messages do the following. With the Communication Builder OFF. Press and hold down the RECORD button while turning the ON/OFF/VOLUME knob ON while keeping the RECORD button depressed, the-unit will start beeping Keep holding the RECORD button until the beeping stops. Turn the unit OFF, wait 5 seconds then turn back ON. Now all previous messages will be erased from the device.