



Folding Steel Frame Commode

Assembly Instructions

1. Remove commode from packaging.

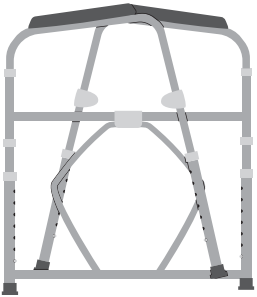

2. Unfold commode and stand upright.

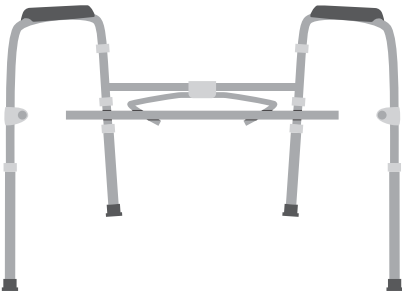

3. Adjust the sides of commode frame perpendicular to the back frame.
4. Attach front cross brace by depressing brass push pins and sliding brace into side frame socket. Attach on both sides. Be sure brass push pins “snap” through holes.



5. Attach back brace by depressing brass push pins and sliding back brace into receptacle located on back of cross brace. Be sure brass push pins “snap” through holes.
6. Lower commode bucket into bracket and attach lid onto back of frame.
7. To fold, depress push pins on back support and remove. Depress push pins on front frame and slide frame from side.

Adjusting Leg Height

1. Before use, adjust height of each leg to your specific needs, making sure all legs are at the same height. When desired height is reached, lock legs into place by aligning buttons into proper holes, per illustration.
2. Ensure legs are locked into place by pushing down or by pulling legs.



Precautions

1. Toilet seat must be in down position before use. Failure to do so may result in breakage of commode and possible injury.
2. Do NOT install commode without first reading and understanding this assembly and instruction sheet. If you cannot understand these instructions, contact a healthcare dealer or technical professional before attempting to install. Otherwise, injury or damage may occur.
3. Inspect all parts for shipping damage before assembly. If there is shipping damage, DO NOT USE. Contact Pisces Healthcare for further instructions.
4. Make certain that the height adjusting snap buttons fully protrude through the same respective hole of each leg extension. This will ensure that the leg extensions are securely locked in position and an even height is achieved.
5. Ensure ALL screws, nuts, and/or bolts are tightened.
6. Check the rubber tips on the leg extensions for rips, wear, or if they are missing. Immediately replace any or all in any of these imperfections exist.
7. All four leg extensions with rubber tips MUST touch the floor simultaneously at all times.
8. Users with limited physical strength should be supervised or assisted while using commode.

Technical support and product questions? Give us a call at 1-888-505-3627.

Distributed by Pisces Healthcare
Longmont, CO 80503
Made in Taiwan