

# Offset Handle Cane

## Safety Summary

To ensure the safe operation of Pisces canes, these instructions **MUST** be followed.

## General Warnings

DO NOT use this equipment without first reading and understanding these instructions. If you are unable to understand the Warnings, Cautions, or Instructions, contact a healthcare professional, dealer, or technical personnel before attempting to use this equipment; otherwise, injury may occur.

Check rubber tip for rips, tears, cracks, or wear. If any of these conditions exist, replace rubber tip immediately. The Pisces Offset Handle Cane can provide ambulatory assistance for an individual weighing up to 300lb (136kg). The cane is designed to provide support, increased stability, and assistance while walking. This cane is NOT designed to support the total weight of individuals.

Ensure that the snap button fully protrudes through adjustment hole of cane. This ensures that the cane is securely locked in position. BEFORE using the cane, ensure that it is securely locked in place or a fall could result, causing bodily injury and/or damage to the cane.

Always consult your physician or therapist to determine proper adjustments and usage.

## Proper Cane Height


For a comfortable fit, ensure shoe height is the same as shoe style worn most frequently when using cane.

1. Stand as upright as possible on a firm, flat surface and let your arm hang down naturally by your side.
2. Adjust cane so top portion of handle touches inside of wrist where wrist meets palm.



## Height Adjustment

1. Turn the anti-rattle collar knob [1] COUNTERCLOCKWISE to loosen.
2. Press IN snap button [2].
3. Slide extension [3] up or down until snap button [2] protrudes through an adjustment hole on handle end of cane. NOTE: An audible "CLICK" will be heard when snap button [2] securely protrudes through the adjustment hole.
4. Repeat Steps 2 - 3 until cane is at height determined PROPER CANE HEIGHT on this instruction sheet.
5. Tighten the anti-rattle collar knob [1] by turning CLOCKWISE.

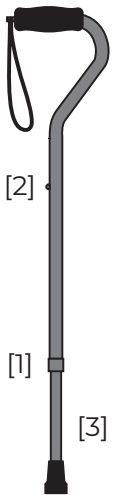

## Maintenance & Care

1. Ensure snap button releases and re-engages properly.
2. Inspect cane tips for wear and damage.
3. Replace any broken, damaged, or worn items immediately.

Technical support and product questions? Give us a call at 1-888-505-3627.

Distributed by Pisces Healthcare  
Longmont, CO 80503  
Made in Taiwan