THERMOPHORE® FREEDOM™ Digital

Moist Heat Therapy Pad

Warranty

Thermophore® products automatically include a two-year limited warranty against manufacturing defects.

Please keep your purchase receipt for warranty eligibility. Altering the product voids the warranty. For questions or assistance visit us at www.battlecreekequipment.com or call us at 269-962-6181.

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PO Box 629
702 South Reed Road
Fremont, IN 46737
269-962-6181
www.BattleCreekEquipment.com

THERMOPHORE® FREEDOM™ Digital

Moist Heat Therapy Pad

Instruction Manual Manuel d'instructions

Digital Switch

Personalized heat & time settings
let you maximize your pain relief

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PO Box 629
702 South Reed Road
Fremont, IN 46737
269-962-6181
www.BattleCreekEquipment.com

Proudly Family
Owned and
Operated!

DANGER

To reduce the risk of burns, electric shock and fire, this product must be used in accordance with the following instructions:

READ THE MANUAL AND FOLLOW ALL WARNINGS AND INSTRUCTIONS.

NEVER USE WHILE SLEEPING.

NEVER USE ON AN INFANT.

THIS PAD IS NOT TO BE USED ON OR BY AN INVALID, PARALYZED PERSON, SLEEPING OR UNCONSCIOUS PERSON, OR A PERSON WITH POOR BLOOD CIRCULATION.

NEVER USE ON AREAS OF INSENSITIVE SKIN.
NEVER TAPE OR TIE DOWN THE SWITCH. THE SWITCH WAS DESIGNED FOR YOUR SAFETY. USE ONLY AS DIRECTED.

BURNS CAN OCCUR REGARDLESS OF CONTROL SETTING, CHECK SKIN UNDER PAD FREQUENTLY.

NEVER USE PAD WITHOUT COVER IN PLACE.

NEVER USE PINS OR OTHER METALLIC MEANS TO FASTEN THIS PAD IN PLACE.

NEVER LAY ON, SIT ON, LEAN AGAINST, CRUSH, OR FOLD PAD DURING USE.

NEVER TWIST THE CORD. EXCESSIVE TWISTING MAY DAMAGE THE INTERNAL COMPONENTS. ALWAYS UNPLUG PAD WHEN NOT IN USE.

NEVER WRAP, HOLD, OR LOOP THE CORD IN ANY POSITION WHICH SHARPLY BENDS IT.

NEVER PULL THIS PAD BY THE CORD. DO NOT USE THE CORD AS A HANDLE. LOOP CORD LOOSELY WHEN STORING. TIGHT WRAPPING MAY DAMAGE CORD AND INTERNAL PARTS.

CAREFULLY EXAMINE INNER COVER BEFORE EACH USE. DISCARD THE PAD IF INNER COVERING SHOWS ANY SIGN OF DETERIORATION.

THIS UNIT IS NOT DESIGNED FOR USE IN CONJUNCTION WITH ANY TYPE OF MASSAGE APPARATUS. SUCH USE MAY DAMAGE THE INTERNAL COMPONENTS, WHICH COULD LEAD TO ELECTRIC SHOCK, BURNS AND/OR FIRE. SUCH USE ALSO WILL VOID THE WARRANTY.

NEVER ATTEMPT TO OPEN, DISASSEMBLE, OR OTHERWISE ACCESS ANY OF THE PRODUCT'S INTERNAL COMPONENTS. THIS WILL VOID THE WARRANTY. IF ANY ISSUE, CONTACT CUSTOMER SERVICE. CONNECT TO A 120V AC OUTLET ONLY.

SAVE THESE INSTRUCTIONS.

Using Your Heating Pad

- Insert the plug into any 120-volt outlet. The pad has a polarized plug as a safety feature so the plug will only fit into a polarized outlet one way.
- **Press and hold the power button to turn on/off the product.**
- Press the SET button to adjust TIME or TEMP. Up ▲ and down ▼ buttons adjust the heat level and timer duration.
- After 10 seconds with no interaction, the switch will exit SET mode and the pad will begin heating.
- Heat bars indicate active heating and when the pad is ready for use.
- Hold the SET button to re-enter SET mode
- The pad will automatically shut off when the timer ends.
- We recommend treatments of 30 minutes or less with a minimum wait between sessions of 60 minutes. Please consult a physician if you require more prolonged heat therapy or if you notice unusual symptoms.

SET Mode

HEATING Mode

Cleaning Instructions

- Do not soak the heating unit. Wipe clean only.
- Hand wash the plush cover in cold water. Hang dry or dry flat.

Storage Instructions

- Store your heating pad flat or in its original packaging in a cool, dark, dry place.
- Do not wrap cord around pad for storage.

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